


Set up sentence tile

Step-by-step instructions for Set up sentence tile in Express Me.

Watch the video, follow the steps, or open the complete transcript for a text version of the instructions.

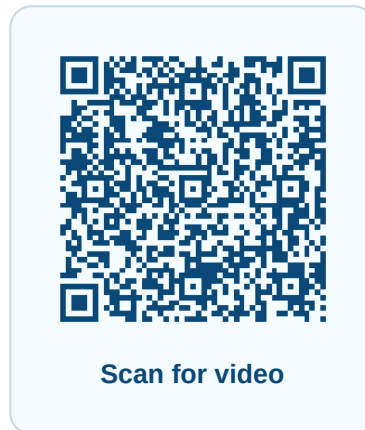

Step-by-step instructions

1

Enter edit mode

Click the button at the top right and confirm **Start editing**.

2

Open the tile menu

Click the three dots on the tile you want to change, for example the **yes** tile.

3

Type a sentence or record speech

Choose an option from the menu. Type a sentence, for example "yes, indeed", or record your own speech.

4

Save the changes

Click **Save** at the top right.

5

Exit edit mode and test the tile

Exit edit mode and click the tile to hear the spoken sentence.

Complete transcript

▼ Read the complete video transcript

Hello everyone, I'm Rick from ExpressMe.

And today I'm going to show you how to set up a sentence for a tile, so that a sentence is spoken instead of the word.

If I for example click on the tile yes now, we hear the following. But we can also set up something different here.

Let's go to edit mode. Click at the top right, then say okay, start editing. And then we can click on the three dots of yes.

Then we see that there are multiple options. We can fill in a sentence or we can record speech ourselves. Now you say for example yes, indeed.

And then we can simply save at the top right. When we exit edit mode and when we now click on yes, we hear the following.

This is a very simple example of course, but you can do all sorts of things with this, so that for example your own name is spoken or longer sentences are spoken.